



1 Put domes on the hearing aids

PICK A PAIR
We recommend starting with the **SMALL** size.

			Closed Domes = more amplification
Lx2	Mx2	Sx2	
			Open Domes = more outside sound
Lx2	Mx2	Sx2	




Hold the end of the hearing aid's tube. Push the dome onto the hearing aid's stem until it feels securely in place.



Pull gently on the dome to make sure it's secure

2 Put the hearing aids on

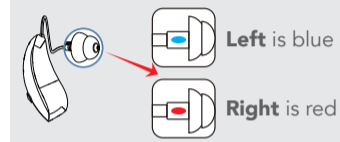


Left is blue
Right is red



Place the body of the hearing aid behind your ear.




Bend the tube so that the dome rests comfortably in the entrance to your ear canal.

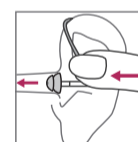


Push the dome into your ear so that it seals the entrance. If it doesn't, the hearing aid might squeal or whistle.


3 Set up the keychain remote


For this step, make sure you have your hearing aids on.

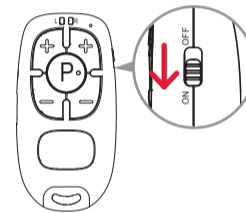


Put the power switch on the side of the remote in the **ON** position.

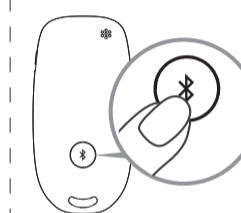

Press the  button on the back of the remote to start pairing with the hearing aids.

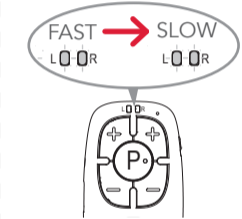

The **L** and **R** indicators flash quickly during the pairing procedure. They flash slowly when the remote and hearing aids are connected.

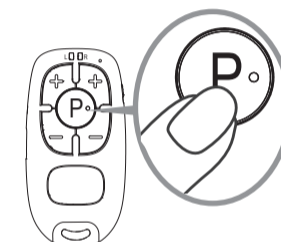


Press and hold the **P** button on the front of the remote until your hearing aids beep. This makes sure your hearing aid settings are in sync with each other.

Note: For maximum battery life between charges, turn the remote off when you take off the hearing aids. **Repeat the remote setup steps when you turn the remote back on.**

4 Choose a preset program

The hearing aids have four preset programs that customize amplification for different environments. Press the **P** button on the remote to change presets. Your hearing aids will beep to tell you which preset you're using.

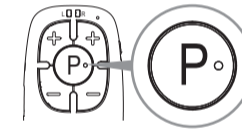


1 Beep
2 Beeps
3 Beeps
4 Beeps

Standard
Restaurant
Outdoors
Music


IMPORTANT: If you notice that your hearing aids are out of sync with each other—for example, if one beeps 2 times and the other beeps 3 when you change presets—press and hold the **P** button on the remote until the hearing aids beep. This puts them back in sync with each other, in preset 1.

5 Adjust the volume

The hearing aids offer 11 levels of amplification. Press the + or - button on the left or right side of the remote to increase or decrease the volume of that hearing aid. The hearing aid beeps. It beeps three times at maximum volume.

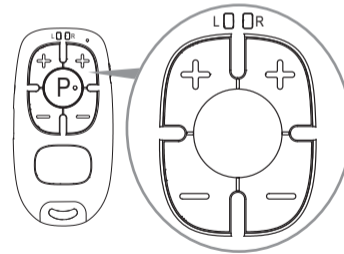


Charge the hearing aids and case

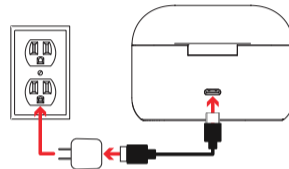

To charge the charging case: Plug one end of the provided charging cable into the back of the charging case. Plug the other end into the charging adapter. Plug the charging adapter into a surge protector or power outlet.

The charging display shows the case's percentage of charge.

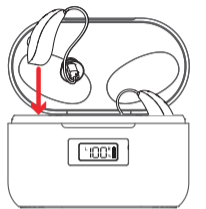


To charge the hearing aids: Insert them in the charging case. Each hearing aid only fits on its side of the case—left on the left side and right on the right.

The case's charging display shows the level of charge of the left (L) and right (R) hearing aids while they're charging.



Charge the keychain remote

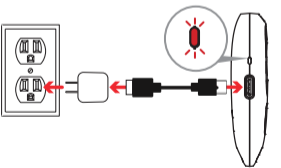

Plug one of the provided charging cables into the charging port on the side of the remote. Plug the other end into the power adapter and plug the adapter into a power source. The charging indicator illuminates.

Tips for new users

- **GIVE IT TIME:** Hearing new sounds might be unsettling at first. This is normal. Your ears and brain need some time to adjust. We strongly recommend giving the hearing aids at least 21 days of routine use before deciding if they're for you or not.
- **PROPER FIT IS IMPORTANT:** Push each dome into your ear canal so that it seals the entrance. If it doesn't, the hearing aid might squeal or whistle. If necessary, pull back on the outside of your ear to widen the entrance to your ear canal while you insert the dome.
- **KEEP AT IT:** Your hearing aids might be uncomfortable at first. This is normal. Start out wearing them a short period of time every day – 30 minutes in a quiet environment. Gradually increase the amount of time you wear them and try them in a wider variety of environments. Eventually they'll start feeling more natural to you.

READ YOUR USER MANUAL to learn about your hearing aids so that you can get the most from them.

For more detailed instructions and information about your new hearing aids, **VISIT RCAHEARINGAIDSUPPORT.COM**



Quick Start Guide

Welcome to the easiest OTC hearing aid to operate! All the controls you need, right at your fingertips. No App! No reaching for your ears! Just use your remote!

This Guide will get you started.